Product Documentation, CANADA ZenGUARD™ Antimicrobial Surgical Mask

Product information

ZenGUARD™ Antimicrobial Surgical Masks | Masques chirurgicaux antimicrobes

Manufactured for / Fabriqué pour: Zentek Ltd., 1123 York Road, Guelph, ON, N1E 6Z1

PRODUCT UPC: 627987885736

English:

ZenGUARD™ Antimicrobial coating helps filter pathogens with bacterial and viral filtration efficiency of >99.9%, providing better protection for healthcare workers and the public.

ASTM LEVEL 3* 4-PLY SINGLE-USE SURGICAL MASKS | Advanced Antimicrobial Agent

	ASTM Level 1	ASTM Level 2	ASTM Level 3	ZenGUARD™ Surgical Masks
Antimicrobial technology	Not a requirement			✓
Viral filtration efficiency	Not a requirement			>99.9%
Bacterial filtration efficiency	≥95%	≥98%	≥98%	>99.9%

ZenGUARD™ antimicrobial Technology, proven to reduce 99.9% of Bacteria**

*Source: ASTM International F2100-20. ZenGUARD™ masks exceed the highest standard specifications for performance of materials used in medical face masks.

**Test completed on E.coli in accordance with AATCC 100-2019 Test Method for Antibacterial Finishes on Textile Material.

WARNING statements:

Masks should not be worn by children under 2 years of age, a person who cannot wear a mask or cannot wear it safely for reasons related to a disability, or a person for whom wearing a mask would create a risk to workplace health, safety, or job duty as determined by the workplace risk assessment.

If the product is damaged or soiled, please do not use.

Directions:

How to wear: Be sure to wash your hands or use hand sanitizer before putting on a mask. Try not to touch the mask when wearing it. Cover your nose and mouth and secure it under your chin. Mask should fit snugly against the sides of your face.

How to remove a mask: Carefully, stretch the ear loops and remove from ears. Handle only by the ear loops or ties. Fold the outside corners together. Be careful not to touch your eyes, nose, and mouth when removing and wash hands immediately after removing.

Ingredients

Active Antimicrobial Agent: ZenGUARD™ (silver-doped graphene oxide)

CAUTION:

Most people with underlying medical conditions could and should wear masks. If you have asthma, you can usually wear a mask. If you have any concerns about the safety of wearing a mask (i.e., you have a respiratory condition), discuss with your healthcare provider the benefits and potential risks of wearing a mask. When the product is in use, please avoid contact with flammable or combustible substances.

Storage:

Store at ambient temperature between 15°C to 30°C.

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